



Medical

LeukoGuard™ LG

Pall Leukocyte Reduction
Arterial Blood Filter for
Extracorporeal Service



AutoVenting Blood Filter

Features and Benefits

- Clear, surface modified polycarbonate housing and leukocyte reducing polyester cartridge
- Low, 220 mL priming volume
- Provides a three-stage system for gas separation with automatic venting at flow rates up to 6 litres a minute
- Includes 40 micron rated polyester screen
- Rapid priming & debubbling
- For arterial and cardiomy lines

Filtration. Separation. Solution.SM



LeukoGuard™ LG

Pall Leukocyte Reduction Arterial Blood Filter for Extracorporeal Service

Filtration Innovation


The history of Pall Medical has been one of noteworthy accomplishments. With the introduction of an extracorporeal leukocyte reduction filter, the LeukoGuard™ LG filter, Pall has entered into a new era of technology and filtration.

...Through Dedicated Air Vent

Air is vented through a dedicated air vent.

A luer lock connection allows the vent to be closed when required.


Performance Data

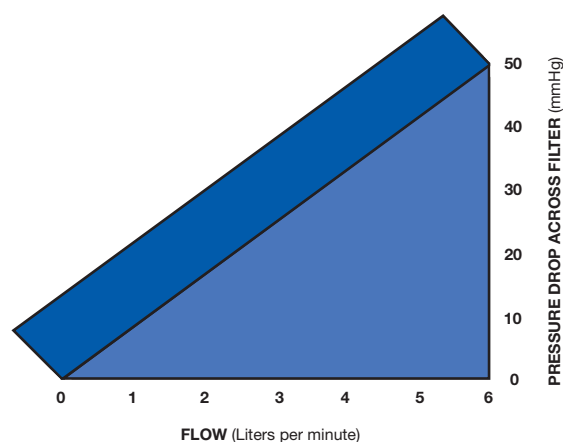

Ordering Information

| Reorder Code | Packaging |
|---------------------------|--------------------------|
| LG6 | 6 filters / case |
| LG6-B (in bypass loop) | 6 filters in loop / case |
| LG6NS* (non-sterile) | 50 filters / case |

Sold only to extra-corporeal tubing packers for use in a custom format.

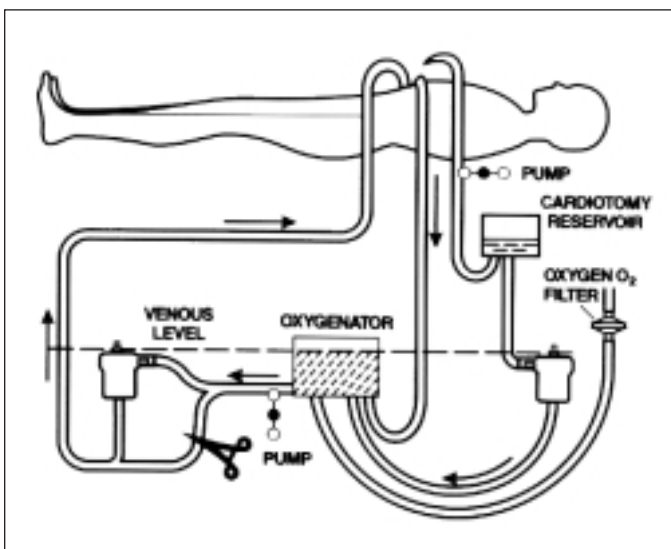

Medical

Manufactured by:

America

2200 Northern Boulevard, East Hills
New York 11548-1289 USA

800.645.6578 toll free
516.484.5400 telephone
516.484.8688 fax

Europe

Europa House, Havant Street,
Portsmouth, PO1 3PD, England

+44 (0)2392 302366 telephone
+44 (0)2392 302505 fax
E-mail: Biosvo@Pall.com



Distributed by:

TERUMO CARDIOVASCULAR SYSTEMS

6200 Jackson Road
Ann Arbor, MI 48103-9300
USA

734 663 4145 telephone
734 663 7981 fax
800 521 2818 toll free

Latin America

TERUMO MEDICAL CORPORATION

Latin American Sales Division
6161 Blue Lagoon Drive, Suite 340
Miami, Florida 33126

305 263 9835 telephone
305 263 1219 fax
800 283 7866 toll free

Europe

TERUMO EUROPE N.V.

Interleuvenlann 40 B-3001
Leuven, Belgium

+32 16 38 12 11 telephone
+32 16 40 02 49 fax

Visit us on the Web at www.pall.com/cv

Select-A-FAX* 800.664.7255

for the latest information delivered to any fax machine, anywhere in the world. This automatic system operates 24 hours a day, seven days a week. Call 1.516.942.0523 from outside the USA.

Select-A-Fax Not available in Europe.

Pall Corporation has offices and plants throughout the world in locations including: Argentina, Australia, Austria, Belgium, Brazil, Canada, China, France, Germany, Hong Kong, India, Indonesia, Ireland, Italy, Japan, Korea, Malaysia, Mexico, the Netherlands, New Zealand, Norway, Poland, Puerto Rico, Russia, Singapore, South Africa, Spain, Sweden, Switzerland, Taiwan, Thailand, the United Kingdom, the United States and Venezuela. Distributors are located in all major industrial areas of the world.

The information provided in this literature was reviewed for accuracy at the time of publication. Product data may be subject to change without notice. For current information consult your local Pall distributor or contact Pall directly. Part numbers quoted above are protected by the Copyright of Pall Corporation Ltd.

PALL and Pall are trade marks of Pall Corporation.
Select-A-Fax is a registered trademark of CyberData Inc.
Filtration. Separation. Solution. is a service mark of Pall Corporation.
©1991,1999, 2000, Pall Corporation.

Printed in England. PMED/1M/CS/08.2002

Filtration. Separation. Solution.SM