



Quick Fit[®]

SAFETY WITHOUT COMPROMISE

**Blood Collection Accessories
Always up-to-date!**



 **TERUMO[®]**

We keep life flowing

Clicking towards safety

The unique bayonet fitting of all QUICK FIT devices significantly facilitates the insertion of needles. Just a small push - and an audible click indicates the tight connection of the blood/urine sampling system.

After use, the needle can be removed without physical contact to it by simply pushing the buttons on the side of the holder. We recommend the use of the QUICK FIT system in combination with TERUMO® TERUSAFE® boxes.

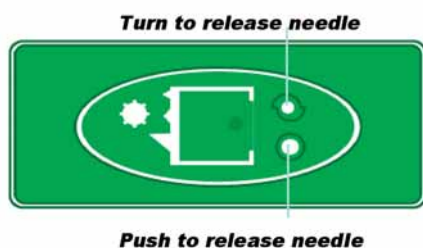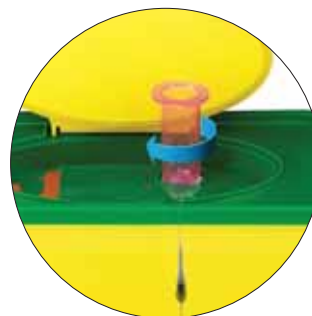

TERUSAFE® boxes are equipped with special disposal apertures perfectly matching the QUICK FIT holder's release technique.

To discard the needle immediately after withdrawal from the vein the QUICK FIT holder only needs to be pushed into this aperture and the needle is automatically released into the TERUSAFE® box.

The QUICK FIT system is designed to ensure the comfort of its users and patients and to complement the outstanding qualities of VENOSAFE™ and VENOJECT® tubes and needles.

Sampling with QUICK FIT holders requires less time than conventional screw systems while the blood collection procedure remains exactly the same.

In brief, QUICK FIT offers you ...

- **Increased safety and efficiency**

As QUICK FIT is very easy to use, speeding up the blood collection process and substantially reducing the risk of needlestick injuries, it can be considered as one of the most cost-effective blood collection systems on the market.

- **No need to learn new procedures**

Sampling with the QUICK FIT system works exactly the same way as with any conventional holder.

- **Improved comfort for patient and phlebotomist**

With the QUICK FIT BacT/ALERT® holder and insert, blood culture samples and venous blood samples can be drawn with one venepuncture only.

- **Less waste and lower disposal costs**

QUICK FIT holders may be reused for at least 150 times, thus reducing the volume of waste and disposal cost.



In order to respond to the market's requirements for safe and quick blood collection and urine sampling, TERUMO expanded its QUICK FIT product range considerably.



Winged Blood Sampling Sets with Surshield® protector and QUICK FIT bayonet or with pre-attached holder

A well developed design allows the Surshield® protector to be put into place with the single motion of one hand, immediately after withdrawing the needle from the patient's vein. You can either connect the winged blood sampling set yourself to the QUICK FIT holder, or you can go for the ready-to-use version, which offers you the winged blood sampling set already pre-attached to a holder.



QUICK FIT: Release holder

One-hand removal

Needle release can be initiated by pressing the two buttons on the sides of the QUICK FIT holder or by pushing the holder into the TERUSAFE® QUICK FIT release aperture. In this way physical contact with the needle is excluded and needlestick injuries avoided.



Needle with special bayonet fitting

The one-step bayonet fitting ensures the secure locking of the needle and prevents the QUICK FIT holder from abrasion.

TERUMO-thin Wall needle

Represents the quality standard in the medical world. The thinner needle wall makes the blood flow more smoothly and faster into the tube, while its sharp, double-bevelled, needle point punctures tissues less traumatically.

QUICK Safe-TE holder: Safety holder

Following the increasing safety awareness, also in blood collection, TERUMO responded to the market's safety requirements by developing an excellent accessory for phlebotomy - the QUICK SAFE-TE holder for multi-sample needles. The QUICK SAFE-TE holder is a comfortable, easy-to-use and ergonomic product designed to facilitate blood drawing and to be safely discarded.



QUICK FIT holder for VENOSAFE™ urine tubes

Representing all the benefits of the QUICK FIT system and clearly distinguishable from blood collection QUICK FIT holders by the yellow colour indication. Blunt QUICK FIT urine-needles allow a quick urine sampling and do not absorb proteins like conventional plastic tubings are likely to do.

VENOSAFE™ Single Use holder

This holder is our solution for any situation where a single-use is required (e.g. by local policy, in infectious disease wards, etc.) and where no disconnection of connected multi-sample needles is possible. It can be used to complete your range of QUICK FIT and QUICK SAFE-TE holders, in order to create a flexible system for blood sampling, depending on your phlebotomist's needs.



QUICK FIT BacT/ALERT® holder

The QUICK FIT BacT/ALERT® holder is especially designed for the use with BacT/ALERT® blood culture bottles. After the blood culture collection the bottle is removed from the holder, the QUICK FIT BacT/ALERT® holder insert is introduced into the holder and the phlebotomist can continue taking venous blood samples with VENOSAFE™ blood collection tubes.



TERUSAFE® boxes

Various sizes of TERUSAFE® boxes are available for the safe release and discard of multi-sample needles, winged blood sampling sets, holders, ..

QUICK FIT SYSTEM

REFERENCES

HOLDERS

CODE	DESCRIPTION	PACKAGING
XX-VF10HGSQ	VENOSAFE HOLDER	40x25



XX-VFHQSAF	VENOSAFE SAFETY HOLDER	10x100
------------	------------------------	--------



XX-VFHBQSU	VENOSAFE SINGLE USE HOLDER	1x1500
------------	----------------------------	--------



XX-ORGHQ	BacT/ALERT® HOLDER	10x10
----------	--------------------	-------



XX-ORGI	BacT/ALERT® INSERT	10x10
---------	--------------------	-------



CODE	DESCRIPTION	PACKAGING
XX-VF10HGSQ4	URINE TUBE HOLDER	40x25



MULTI-SAMPLE NEEDLES

CODE	DESCRIPTION	PACKAGING
MN-2038MQ	0.9 x 40 mm	20x100
MN-2025MQ	0.9 x 25 mm	20x100
MN-2138MQ	0.8 x 40 mm	20x100



MN-2125MQ	0.8 x 25 mm	20x100
MN-2238MQ	0.7 x 40 mm	20x100
MN-2225MQ	0.7 x 25 mm	20x100

URINE SAMPLER

CODE	DESCRIPTION	PACKAGING
UN-2038Q	0.9 x 40 mm (blunt)	20x100



VENOJECT LUER-ADAPTER

CODE	DESCRIPTION	PACKAGING
XX-MN2000Q		20x100



DISPOSAL BOXES

CODE	DESCRIPTION	PACKAGING
XX+Q2500	TERUSAFE DISPOSAL BOX 2.5L, RECTANGULAR	45
XX+Q2510	TERUSAFE DISPOSAL BOX 2.5L, ROUND*	45
XX+Q2010	TERUSAFE DISPOSAL BOX 2L, ROUND*	60
XX+Q1510	TERUSAFE DISPOSAL BOX 1.5L, ROUND*	80
XX+Q0001	TERUSAFE ADHESIVE ADAPTER	50

* can be used with adhesive adapter



TÜVIGS approved

WINGED BLOOD SAMPLING SET

CODE	TUBING	PACKAGING
MN-SV21Q	300 mm	16x50
MN-SV23Q	300 mm	16x50

with Surshield® protector

CODE	TUBING	PACKAGING
MN*SVS21BQ30	300 mm	16x50
MN*SVS23BQ30	300 mm	16x50
MN*SVS21BQ18	180 mm	16x50
MN*SVS23BQ18	180 mm	16x50
MN*SVS21BQ09	90 mm	16x50
MN*SVS23BQ09	90 mm	16x50



with pre-attached Holder and Surshield® protector

CODE	TUBING	PACKAGING
MN-SVS21H18	180 mm	12X40
MN-SVS21H30	300 mm	12X40
MN-SVS23H18	180 mm	12X40
MN-SVS23H30	300 mm	12X40

with pre-attached Holder

CODE	TUBING	PACKAGING
MN-SV21H18	180 mm	12X40
MN-SV21H30	300 mm	12X40
MN-SV23H18	180 mm	12X40
MN-SV23H30	300 mm	12X40

Biological samples and blood collection sharps, in particular, should be considered as potentially infectious and should, therefore, be handled with care. All relevant instructions for use, as well as the policies and procedures of your facility, should be followed closely. Never recap used needles, but dispose them into approved biohazard containers.

Every day, as healthcare professionals, you are working to ensure life and to contribute to the well-being of your patients. Every day, Terumo Europe provides you with the clinical systems, means and solutions to keep life flowing.



TERUMO®

We keep life flowing

TERUMO CORPORATION
44-1, 2-chome Hatagaya Shibuya-ku Tokyo 151-0072 JAPAN
Tel. +81/3 3374.8111 Fax +81/3 3374.8196

TERUMO EUROPE N.V.
Researchpark Zone 2 Haasrode Interleuvenlaan 40 B-3001 Leuven BELGIUM
Tel. +32/16.38.12.11 Fax +32/16.40.02.49

TERUMO EUROPE N.V. BENELUX SALES DIVISION
Researchpark Zone 2 Haasrode Interleuvenlaan 40 B-3001 Leuven BELGIUM
Tel. +32/16.39.25.80 Fax +32/16.39.25.99 THE NETHERLANDS
Tel. 0800/022.03.96 Fax 0800/022.04.14

LABORATOIRES TERUMO FRANCE S.A.
1, Parc Ariane Hatagaya Shibuya-ku Tokyo 151-0072 JAPAN
Tel. +33/1 30.96.13.00 Fax +33/1 30.43.60.85

TERUMO DEUTSCHLAND GmbH
Hauptstrasse 87 65760 Eschborn GERMANY
Tel. +49/61.96.80.230 Fax +49/61.96.80.23.200

TERUMO DEUTSCHLAND GmbH
Zweigniederlassung Wettingen Bahnhofstrasse 88 CH-5430 Wettingen SWITZERLAND
Tel. +41/56.419.10.10 Fax +41/56.419.10.11

TERUMO EUROPE ESPAÑA SL
Avda. Juan Carlos I, N° 13 - 7ª Planta Edificio Torre La Garena 28806 Alcalá de Henares (Madrid) SPAIN
Tel. general: +34/902.10.12.98 Tel. cust. serv.: +34/902.20.31.40 Fax general: +34/902.10.13.58 Fax cust. serv.: +34/902.10.17.57

TERUMO ITALIA SRL
Via Simone Martini 143/145 I-00142 Roma ITALY
Tel. +39/06.519.61.420 Fax +39/06.503.04.07

TERUMO SWEDEN AB
Nya Varvet, Byggnad 90 Sven Källfelts Gata 18 SE-426 71 Västra Frölunda SWEDEN
Tel. +46/31.748.58.80 Fax +46/31.748.58.90

TERUMO DENMARK Filial af TERUMO SWEDEN AB
Tel. +45/7020.93.80 Fax +45/7020.94.80

TERUMO UK Ltd.
Tamesis The Causeway Egham Surrey TW20 9AW UNITED KINGDOM
Tel. +44/1784.476.200 Fax +44/1784.476.201

TERUMO EUROPE N.V.
Moscow Representative office 4, Ivana Franko St 121108 Moscow RUSSIA
Tel. +7/495.234.3303 Fax. +7/495.234.3303

® Registered Trademark Published by Terumo Europe N.V.

Addresses are subject to change, please consult our website.

www.terumo.com
www.terumo-europe.com