



# Capiject<sup>®</sup>

Capillary blood sampling system



**Accurate  
and easy  
sampling**




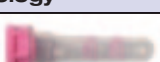
- Extra wide collection lip for enhanced specimen flow
- Silicone coating for easy transfer of blood to the bottom of the tube
- Easy opening of tube due to ridged cap
- Superior olefin-based gel for clean separations
- Tube stays in cap allowing tube to stand alone
- Certified for lead testing and latex-free
- Colour coding of cups for fast recognition

# Capiject® system with collection lip

The extra large collection lip of the Capiject tube supports the specimen flow. The sleek, tapered silicone-coated interior additionally reduces sticking and allows an easy transfer of

the blood to the bottom of the tube. The colour coded fill-lines enable accurate sampling and the ridged cap facilitates opening and closure. The superior olefin-based gel provides

a cleaner separation and makes a wider range of clinical testing possible. The easy handling accelerates the collection process and lowers the risk of contamination.

Product name	Code	Volume	Additive	Centrifugation	Shelf-life
<b>Biochemistry (Serum)*</b>					
	T-M	500 µl	None	1200 G/10 min 3500 G/90 sec	24 months
<b>Biochemistry (Serum Gel)*</b>					
	T-MG	500 µl	Gel - Clot activator (silica particles)	1200 G/10 min 3500 G/90 sec	24 months
<b>Biochemistry (Serum Gel - Bilirubin)</b>					
	T-MGA	500 µl	Gel - Clot activator (silica particles)	1200 G/10 min 3500 G/90 sec	24 months
<b>Biochemistry (Plasma)*</b>					
	T-MLH	500 µl	Lithium Heparin 12.5 I.U.	1200 G/10 min 3500 G/90 sec	24 months
<b>Biochemistry (Plasma Gel)*</b>					
	T-MLHG	500 µl	Gel - Lithium Heparin 12.5 I.U.	1200 G/10 min 3500 G/90 sec	24 months
<b>Glycaemia*</b>					
	T-MPS	250 µl	Sodium Fluoride/ Potassium Oxalate	1200 G/10 min 3500 G/90 sec	24 months
<b>Hematology*</b>					
	T-MQK	500 µl	K2-EDTA 0.78 mg/tube	1200 G/10 min 3500 G/90 sec	24 months

Packaging: 50 tubes per bag – 5 bags per box (250) – 4 boxes per case (1000)

(\*certified for lead testing)

 **Tubes containing an additive have to be inverted 8-10 times immediately after collection!!!**

Every day,  
as healthcare  
professionals,  
you are working  
to ensure life and  
to contribute  
to the well-being  
of your patients.  
Every day,  
**Terumo Europe**  
provides you  
with the clinical  
systems, means  
and solutions  
to keep life flowing.



#### TERUMO CORPORATION

44-1, 2-chome  
Hatagaya  
Shibuya-ku  
Tokyo 151-0072  
JAPAN  
Tel. +81/3 3374.8111  
Fax +81/3 3374.8196

#### TERUMO EUROPE N.V.

Researchpark Zone 2 Haasrode  
Interleuvenlaan 40  
B-3001 Leuven  
BELGIUM  
Tel. +32/16.38.12.11  
Fax +32/16.40.02.49

#### TERUMO EUROPE N.V.

**BENELUX SALES DIVISION**  
Researchpark Zone 2 Haasrode  
Interleuvenlaan 40  
B-3001 Leuven  
BELGIUM  
Tel. +32/16.39.25.80  
Fax +32/16.39.25.99  
THE NETHERLANDS  
Tel. 0800/022.03.96  
Fax 0800/022.04.14

#### LABORATOIRES TERUMO

**FRANCE S.A.**  
1, Parc Ariane  
Bâtiment Uranus  
Rue Hélène Boucher  
78284 Guyancourt Cedex  
FRANCE  
Tel. +33/1 30.96.13.00  
Fax +33/1 30.43.80.85

#### TERUMO DEUTSCHLAND GmbH

Hauptstrasse 87  
65760 Eschborn  
GERMANY  
Tel. +49/61.96.80.230  
Fax +49/61.96.80.23.200

#### TERUMO EUROPE ESPAÑA SL

Edificio Torre La Garena  
Avda. Juan Carlos 1º Planta 7ª  
28806 Alcalá de Henares (Madrid)  
ESPAÑA  
Tel. general: +34/902.10.12.98  
cust. serv.: +34/902.12.31.40  
Fax general: +34/902.10.13.58  
cust. serv.: +34/902.10.17.57

#### TERUMO ITALIA SRL

Via Simone Martini 143/145  
I-00142 Roma  
ITALIA  
Tel. +39/06.519.61.420  
Fax +39/06.503.04.07

#### TERUMO SWEDEN AB

Nya Varvet  
Byggnad 90  
SE-426 71 Västra Frölunda  
SWEDEN  
Tel. +46/31.748.58.80  
Fax +46/31.748.58.90

#### TERUMO DENMARK

**Filial af TERUMO SWEDEN AB**  
Tel. +45/7020.93.80  
Fax +45/7020.94.80

#### TERUMO UK Ltd.

Tamesis  
The Causeway  
Egham  
Surrey  
TW20 9AW  
UNITED KINGDOM  
Tel. +44/1784.476.200  
Fax +44/1784.476.201

#### TERUMO EUROPE N.V.

Moscow Representative office  
4, Ivana Franko St  
121108 Moscow  
RUSSIA  
Tel. +7/495.234.3303  
Fax. +7/495.234.3303

®: Registered Trademark  
Published by Terumo Europe N.V.

[www.terumo.com](http://www.terumo.com)  
[www.terumo-europe.com](http://www.terumo-europe.com)

 **TERUMO®**

We keep life flowing