

Quality time for better care


"Quality time for better care" is the brand promise of Terumo Medical Care Solutions.



HOSPITAL CARE

AGANI™

Safety Hypodermic Needle

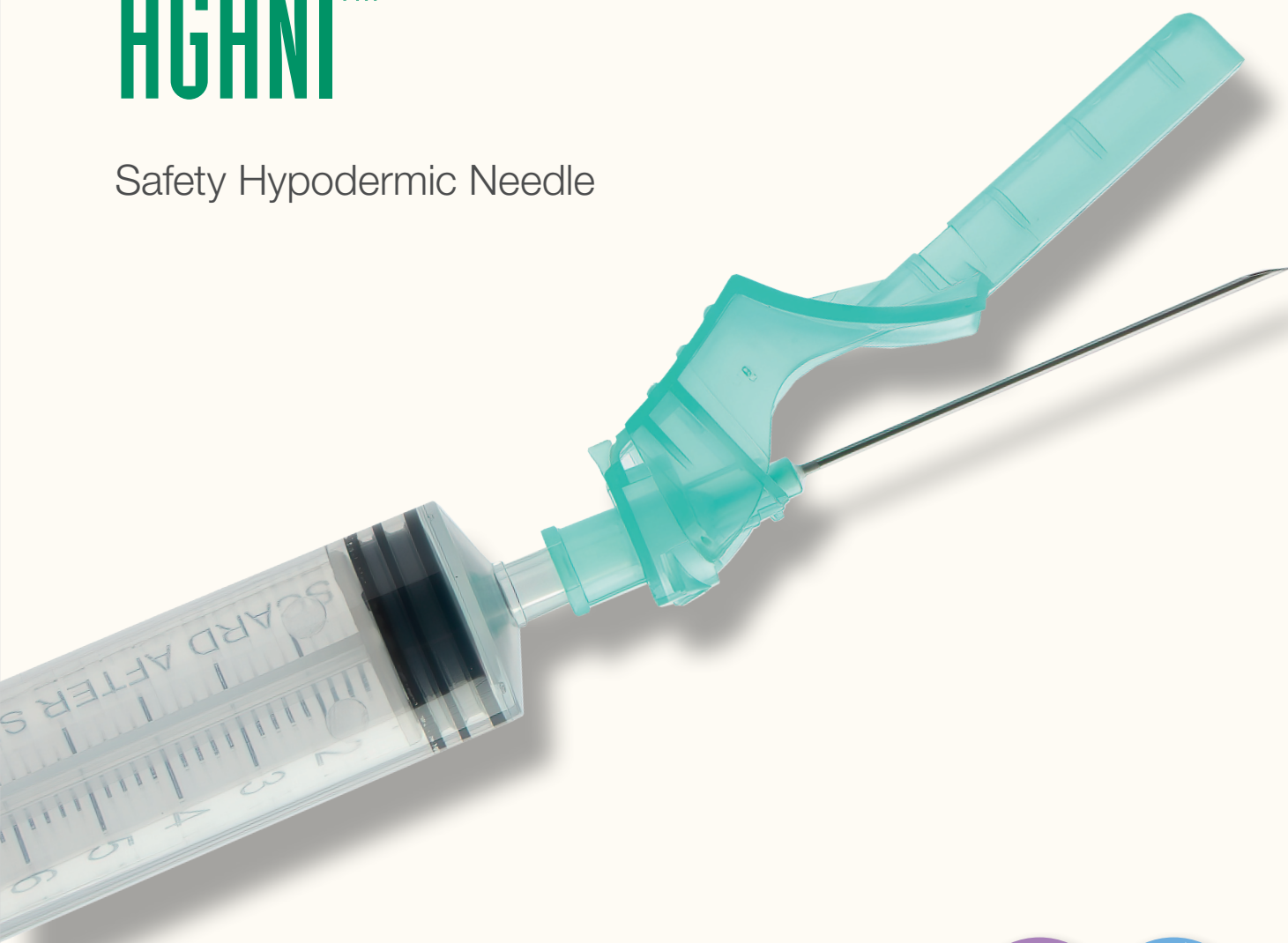

Avoid a needle stick
with a simple click


Managing the risk with Agani™ Safety Hypodermic Needles

The more injections - the higher the risk of sharps injuries
Increasing the need for safety needles



Agani™ Safety Hypodermic needles deliver a solution to prevent needle stick injuries. After withdrawing the needle, the attached safety shield can be activated to cover the needle and minimize the risk of an accidental needle stick.

Activation of the safety mechanism

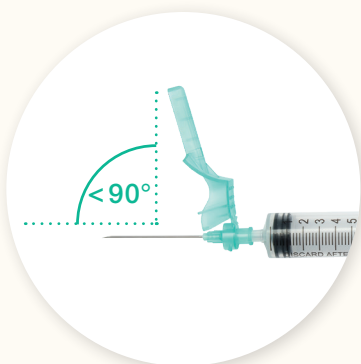

Use one hand technique and push the tab forward with your finger or thumb so that the sheath is $<90^\circ$ from the needle. Keep your finger or thumb behind the tab at all times.



Position the sheath approx. 45° to a flat surface.



Press down with a firm, quick motion until you hear a distinct audible click.

Helping to prevent healthcare worker injuries and needle stick infections



All needle stick injuries have risk

- They may carry dangerous pathogens such as Hepatitis B virus, Hepatitis C virus or HIV
- They cause unnecessary anxiety and stress to healthcare workers
- The prevention of worker's injuries caused by medical sharps is covered under the European Directive 2010/32/EU on the prevention from sharps injuries in the hospital and healthcare sector¹

Avoid the risk of severe infections!



Injection & Access

**Delivering
sharps protection**

Secure and easy handling protects healthcare workers at risk

The use of safety devices has been proven beneficial in the prevention of sharps injuries in the hospital and healthcare sector.^{1,2}



Versatility
Can be fitted with standard Luer Slip or Luer Lock syringes.

ISO 6009 colour-coded hub
easy identification of the required size by the ISO colorcodes.

Secure safety activation «CLICK»
Audible feedback and clear sheath window ensure confirmation of secure locking of the needle in the shield.

One-hand activation
Safety mechanism allows sheath activation with one hand.



Smooth and easy puncture increases application comfort

Agani™ Safety Hypodermic needles have a sharp lancet tip and an outside layer of silicone lubricant.

The sharp point

- Facilitates the penetration of the skin
- Helps to minimize the trauma

The silicone coating

- Provides good gliding properties
- Allows a smooth insertion

Agani™ Safety Hypodermic needles are designed to enable high levels of safety, care and comfort in the injection or aspiration of drugs or blood. A large portfolio makes them suitable for a wide range of medical applications.

Choose safety throughout the range



Agani™ Safety Hypodermic Needles


from 18G up to 30G


Single use, latex-free, EO sterilized hypodermic needles, made from plastic materials and stainless steel.


All needles come sterile, individually packed in colour coded blister packaging.





The range includes a broad variety of needle lengths and diameters


 **18G Cannula (ø 1.2 mm)**
Length 25, 38 mm
Colour Pink


 **19G Cannula (ø 1.1 mm)**
Length 25, 38 mm
Colour Cream


 **20G Cannula (ø 0.9 mm)**
Length 25, 38 mm
Colour Yellow


 **21G Cannula (ø 0.8 mm)**
Length 25, 38 mm
Colour Green

 **22G Cannula (ø 0.7 mm)**
Length 25, 38 mm
Colour Black

 **23G Cannula (ø 0.6 mm)**
Length 25, 38 mm
Colour Blue

 **25G Cannula (ø 0.5 mm)**
Length 16, 25, 38 mm
Colour Orange

 **26G Cannula (ø 0.45 mm)**
Length 13 mm
Colour Brown

 **27G Cannula (ø 0.4 mm)**
Length 13 mm
Colour Grey

 **30G Cannula (ø 0.3 mm)**
Length 13 mm
Colour Yellow

For further ordering information, please refer to the Agani™ Safety Hypodermic needles product information card.



Dedicated to making things better

Sharps protection for healthcare workers provides an important tool in healthcare setting.

Our goal is to create simple, effective solutions to support all those giving their best to treat patients.



**Contributing to Society
through Healthcare**



TERUMO EUROPE N.V.

Terumo Corporation Terumo Europe N.V.
+81 3 6742 8500 +32 16 38 12 11

www.terumo-europe.com

Your Terumo partner

All brand names are trademarks or registered trademarks owned by TERUMO CORPORATION, its affiliates, or unrelated third parties. Published by Terumo Europe N.V.

Please note that some products may not be available in all regions, check with your local Terumo contact.